ENG

MAMI

Baby's health, Our pursuit

All-in-one Smart Electric Breast Pump

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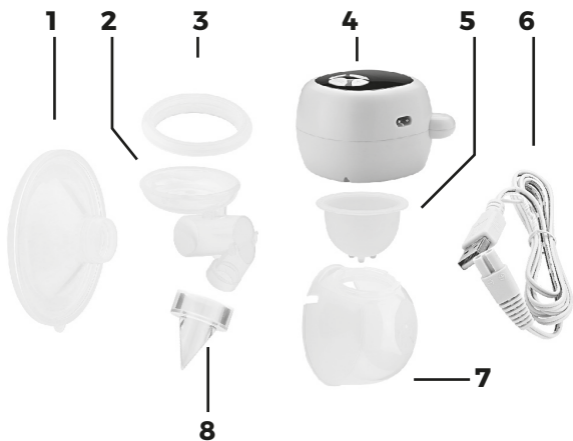
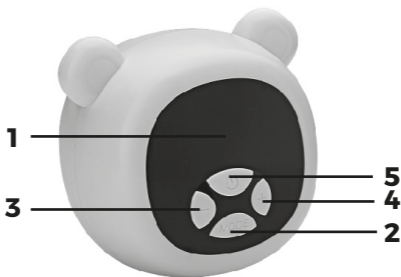
- 5.** Silicone valve
- 6.** USB line
- 7.** Milk collector
- 8.** White valve

Charging time: 4 hours

Using time: 3 hours

Weight: 290g (main body)

Rated voltage: 5V, 1A

PRODUCT DESCRIPTION

- 1.** HD LCD display.
- 2.** Mode button, short press to switch function made.
- 3.** Reduce vacuum.
- 4.** Increase vacuum(6 levels in each made).
- 5.** Power button, press for 3 seconds to tum on/off, press once to enter the working state of each made, press aga in to pause werk, long press for 3 seconds to shut down.

CLEAN AND STERILIZE

- 1.** Before use, you must disassemble, wash, and sterilize all the breast pump parts that come inte contact with breast milk.
- 2.** Drinking water recommended for cleaning.
- 3.** Remove and clean the parts that come inte contact with breast and breast milk immediately after use to avoid bacteria breeding caused by residual breast milk.

WARNING!

Never wash or sterilize the pump motor, USB line. You can wipe those parts clean with a soft cloth

Take care when cleaning the white valve, silicone diaphragm and silicone valve. If them get damaged, your breast pump will not function properly.

Clean the white valve, do not knead. Do not insert anythings, or it may cause damage

ASSEMBLING

To assemble the electric breast pump, fellow the steps as below.

1. Put the silicone diaphragm into the component connector, and press it with your hands to fasten the edges firmly to ensure a complete seal.

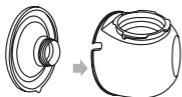
2. Install white valve into the bottom of the component connector, make sure it is installed in the correct direction



3. Put the assembled component connector into the milk collector, with the „ear” part of the linker aligned with the internal slot of the milk collector, and press the linker disc part with both hands to fasten the edges firmly to ensure a complete seal.

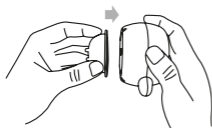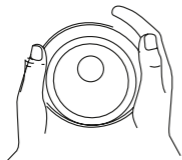4. Install the silicone shield on the milk collector, and press the edges with both hands to ensure that it fits into place.

5. Insert the silicone valve into the pump body from underneath. Make sure it fits securely around the rim by pressing down with your fingers to ensure a perfect seal. (Note: The silicone valve should fit tightly to the bottom of the pump body. Otherwise, the breast pump will not work properly.)

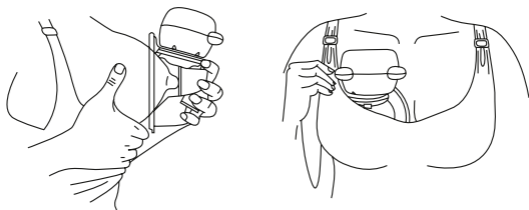
6. With the silicone diaphragm facing down, screw the pump body to the component connector until it is firmly fixed

7. Connect the USB charging line to charge the pump motor.

WARNING

The power adapter is a common standard accessory in the market. It is not included in this product, if necessary, you may purchase yourself. Power adapter specification:5V,1A

HOW TO USE

To begin pumping

1. Press the silicone shield firmly against the breast, leave no gaps and make sure your nipples are centered. Keep it upright. Be careful not to distort the position of the product and the chest, otherwise the breast pump will not have suction.
2. Press the power button for 3 seconds to start the breast pump.
3. You can press the mode button to select the mode you need. There are 4 modes (massage mode, stimulation mode, pumping mode and fast pumping mode), each mode has 6 levels of suction. You can choose the suction that makes you comfortable by pressing the „+“ and „-“ button. In the early postpartum period, the milk duct is not easy to conduct, and massage mode can be used to stimulate the breast and effectively promote breast milk secretion.
4. You can press the mode button to select the pumping mode. There are 6 levels in this mode. You can select the suction that is comfortable for you by pressing the „+“ and „-“ button lightly. In the early postpartum period, breast discomforts such as swelling and lumps are prone to occur during breastfeeding. This mode can alleviate breast problems.

5. The machine has a memory function, When you turn it on again, it will work as your last working mode and levels.
6. After finishing pumping, press and hold the power button for 3 seconds to turn off the breast pump, and then carefully remove the breast pump from the breast.

BREAST MILK STORE AND THAW

CAUTION

1. Never refreeze breast milk.
2. Do not mix fresh milk breast milk to already frozen milk.
3. Always discard and left over milk at the end of a feed
4. Only store the breast milk collected by sterilized breast pump.

STORING YOUR BREAST MILK

Please store the breast milk with the clean and sterilized bottle or the milk storage bag, and freeze it in the fridge.

The milk can be kept in the refrigerator for 48 hours (one the inside of the refrigerator) or in the freezer for three months. Label the bottle or storage bag with the date and time. The most early milk should be use first.

STORING YOUR BREAST MILK

You can keep frozen breast milk over 0 degree one night for thawing.

In an emergency, you can use bot water or milk warmer machine to help you unfreeze the milk. Milk should be fed your baby in 24 hours after thawing.

How to warm up the breast milk? Vou can put it into the milk bottle and use milk warmer machine to warm up in a perfect temperature.

CAUTION

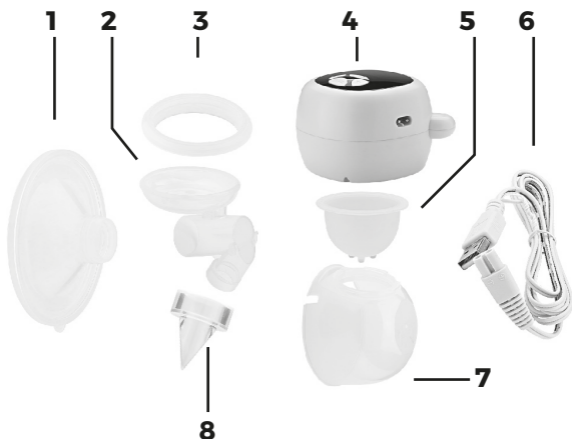
1. Remember! Always check milk or food temperature before feeding.
 2. Never heat breast milk in microwave as this may destroy valuable nutrients and antibodies.
 3. In addition, uneven heating may lead to local overheating of the liquid, be careful of it.
 4. Never immerse a frozen bottle in boiling water to avoid crack or split.
-

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