# ELECTRIC NASAL & EAR ASPIRATOR

SAFE FOR NEWBORS AND INFANTS

---

Soft sllicone tipped infant tweezers for effective removal of nasal discharge earwax

Specially designed with suitable suction pressure for newborn infants.

**3 seconds pressing start.**

## 1. TABLE OF CONTENT

---

**Thank you for choosing our products.  
It is more convenient and safer if you read the  
user's manual before using the product.**

<b>1. Table of Contents</b>	<b>2</b>
<b>2. Cautions for Safety</b>	<b>3</b>
<b>3. Name of Each Part</b>	<b>4</b>
<b>4. Preparation prior to use</b>	<b>5</b>
<b>5. How to Use</b>	<b>6-7</b>
<b>6. Storage &amp; Maintenance</b>	<b>8</b>
<b>7. What to Check upon Mechanical Failure</b>	<b>9</b>

Please be sure to read the user's manual before use for  
proper usage of the product  
This product is approved for medical device.

## 2. CAUTIONS FOR SAFETY

---

Please comply with „Safety Warnings“ since it is aimed to prevent any safety accidents or dangers by the proper and safe use of the product. „Safety Warnings“ are composed with „Warning“ and „Caution“ where the definition of „Warning“ and „Caution“ is as follows:

 **WARNING**

Potential hazardous situation where critical casualty or serious injury may occur if not avoided.

 **CAUTION**

Potential hazardous situation where slight injury may occur if not avoided.

This symbol signifies that a special attention is required upon potential danger under the special circumstances. Please be sure to comply with the guidelines through thorough understanding in order to prevent dangerous situation.

 **WARNING**

1. For those who with the following conditions, please consult with your doctor before using the product.

- Those who with heart condition, malignant tumor and require to rest
- Those who with heart disorder, delicate skin, sensitiveness or other skin ailments
- Those who with mental disorder, nasal scar, frequent swelling of the body or high fever
- Those who under doctor's care

due to other physical disorders or ailments or are in use of the product on the lesion of current ailment or the previously treated lesion

- Those who currently under doctor's care due to ailments or other allergies on nose, eyeball or ear or with a possibility of imminent doctor's care

2. Do NOT use the product for any purpose other than specified by the device.

 **CAUTION**

Be sure to avoid the overflow of the suction container and to dean it after its use.

2. Please be sure to tum the power off when separating the aspirator from the body.

3. Please check if the device normally functions after checking the contact status of the switch.

4. Please be cautious of the followings during the use of the device.

a) Check the status of the device and if the patient experiences any anomaly while in use.

b) When anomaly is detected from the device or patient, please stop the operation of the device, make sure of safety of the patient and take necessary measures.

5. Please be sure to be fully familiar with the user's manual before using the device.

6. Please be cautious of the followings upon storage.

- a) Please keep the product away from children.
- b) Please use and keep the product on dry location free from water.
- c) Keep the product at the location free from any harmful influence by air including air pressure, temperature, humidity, ventilation, dust, wind, salt or other contaminants.
- d) Be aware of slopes, vibration or shocks.
- e) Avoid chemical storage place or the location with gas.
- f) Avoid any place with the source of heat generation such as fireplace, radiator, stove, TV or other home appliances.
- g) Maintain the device clean for the next use.

7. Please contact customer service center upon functional failure after diagnosing the status, and DO NOT disassemble or assemble the device at the customer's own discretion.

8. Please do NOT alter the device.

9. Maintenance

- a) Be sure to check the device and its part on regular basis.
- b) Avoid using for long hours, and be sure to check the device for normal operation before reusing the product that has been in storage.

Please prevent children from playing with the device. individual parts of the device might get stuck on the throat. Please consult with your doctor how to use the product and how frequently to use the product on child's nose. Please do NOT place the device too deep into the nose. Do NOT press the tip of the device too hard against the nasal wall.

Do NOT apply the device to ear, mouth or eye.


This aspirator is a device designed to relieve nasal congestion. Do NOT use the device for any other purpose than specified.

When using the tweezers for infant, be sure to use with the tweezers-tip; otherwise, it poses a potential danger for injury.

---

### 3. NAME OF EACH PARTS

#### 1. BODY

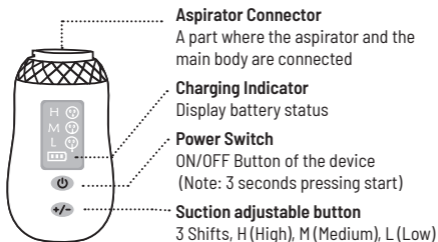## 2. ASPIRATOR

### Suction tips

(A) A tip for aspiration of runny secretion

(B) A tip for aspiration of soft secretion

(C) A tip for aspiration of ear secretion

**Main body of aspirator**  
A case protecting the contents of aspirator

### Suction container cap

A cap to prevent the back flow of the secretion contained

**Suction storage container**  
Storage place for secretions contained


### Connector for Aspirator Main Body

A part where the aspirator is connected to the main body

## 3. TWEEZERS

### Tweezers tip

Protective tip to protect the nasal mucous membrane and to prevent deep insertion.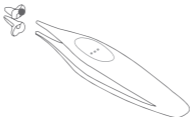


### Tweezers for infants

Tweezers for the removal of solid secretion inside nasal cavity.

## 4. USERS MANUAL


### 4. PREPARATION PRIOR TO USE

- Please check the followings after carefully taking the product out of the box.
  - Please check thoroughly for any damages to the product.
  - Please check if there is any missing part.
  - Please contact the place of purchase or sales agency if any damages or missing parts are discovered.
- Charge the appliance fully before you use it for the first time.
  - To charge the appliance, insert the small plug into the socket on the back of the appliance, and connect the other end of the USB cable to the power adapter. (Charging head is not included).
  - The appliance is charging when the battery conditions is moving. Charging takes up to 3 hours.
  - The appliance is fully charged when the charging indicator light stopped twinkling, operating time is up to 60 minutes.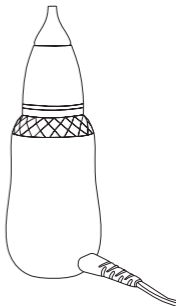


## 5. HOW TO USE


### 1. How to assemble the aspirator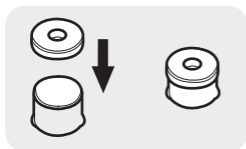


- 1** Place the suction container cap upon the suction container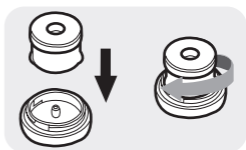

- 2** Put the suction container into the connector on the main body of the aspirator. Turn to the direction of the arrow.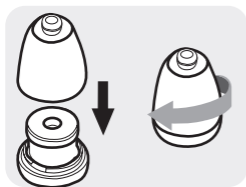

- 3** Put and twist the aspirator main body for a firm fixation.

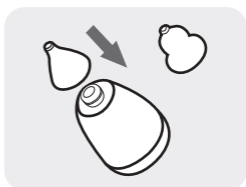- 4** Put the suction tip into the main body of the aspirator.

### 2. Put aspirator into the main body.

Place a fully assembled aspirator into the connector on the main body of aspirator with the groove aligned. Turn to the direction of the arrow.

Please select suction tip is appropriate for the size and shape of child's nose. At the same time, congestion condition of the child should also be considered.

**Suction Tip (A):**

A tip for aspiration of runny secretion

**Suction Tip (B):**

A tip for aspiration of soft secretion

**Suction Tip (C):**

A tip for aspiration of ear secretion




**3. Press and hold the power switch for 3 seconds until the appliance work and then loosen your finger.**

- Please make sure the aspirator connected to the main body before turning the power on.
- This product is NOT a toy.
- Please be sure NOT to let children play with the product. Each part poses a danger as it may get caught on throat.

**4. Press the suction adjustment button to adjust to the appropriate suction.**

For children unable to use the device by themselves, please use the product with proper suction while the child is lying or being held comfortably.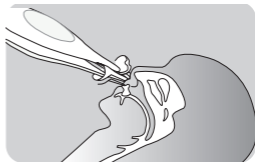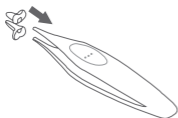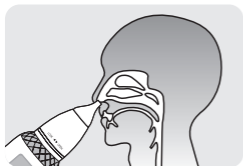


- For children unable to use the device by themselves, please use the product with proper suction while the child is lying or being held comfortably. Please do NOT place the device too deep into the nose.
- Do NOT press the tip of the device too hard against the nasal wall. Do NOT apply the device to mouth or eye. This aspirator is a device designed to relieve nasal congestion. Do NOT use the device for any other purpose than specified.
- Remaining foreign substance should be removed with the tweezers for infant. Be sure to use the tweezers for infant with the tweezers-tip. Tweezer's tip protects the nasal mucous membrane and prevents deep insertion into the nose. When using the tweezers for infant, be sure to use with the tweezers-tip.
- Tweezers pose a potential danger for injury without its tip on.

**5. Turn the power off by pressing the power switch after completing its use.**

## 6. STORAGE & MAINTENANCE

1. Disassemble the aspirator by separating the aspirator from the main body after the use.

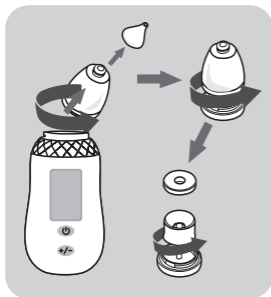2. Discard the foreign substance collected in the suction container.

3. Please sanitize the disassembled aspirator parts with boiling water or clean them thoroughly with running water.

4. Dry thoroughly after cleaning.

### CAUTION

Please be sure to avoid the parts from touching the hot inner surface of the pot when boiling them with water.

5. Wipe the main body with soft cloth or clean it with water-soaked cloth if necessary.

### CAUTION

Please do not put the main body into water or other solutions

Nasal discharge with low viscosity might flow backward at times. Upon such occasion, please clean the surrounding area of the absorption hole of the main body by using cotton swab.

\*Upon back flow, it may cause malfunction due to excessive operation of the motor. Upon such occasion, please contact customer service center for the cleaning of the motor.

## Storage & Maintenance after-the-use-of-the-product

1. Please avoid using or storing the product where fluid may come in contact with the product.
2. Upon storage, please avoid the place with high humidity, temperature, sunlight or bad ventilation or air; and place the product at the place without shock or vibration.
3. Upon mechanical failure, please ask customer service center without disassembling the product.
4. Please check the parts of the device on regular basis.

---

## 7. CHECKLIST UPON MECHANICAL FAILURE

PROBLEM	CAUSE	SOLUTION
When suction is too weak or not working.	The aspirator is with low battery.	Check the charging indicator.
	Assembly is inappropriately conducted.	Check the assembly of the aspirator (See Page 6)
	The connection between aspirator and main body is unstable.	Check the connection between aspirator and between aspirator main body.

---

## PRODUCT SPECIFICATIONS

Name of Product	Electric Nasal & Ear Aspirator
Rated Voltage	5V 1000mA
Weight	165g (Main body)
Packing Unit	1 set
Purpose of the Use	A device used to absorb nasal, ear secretion and mucus by utilizing external vacuum pressure

# ELECTRIC NASAL & EAR ASPIRATOR

SAFE FOR NEWBORNS AND INFANTS

My baby still has a runny nose even after visit to doctor's office. Poor baby... I can't afford going to see doctors every time. Is there any other way?


Aspirator will wipe away all the worries and stress from you and your baby. Vancocon effectively cleans and absorbs nasal secretion from the deepest part of a nose without any stimulus.

..... **1** Lay down or hold your baby safely.

**2** Put the silicone tip all the way into your baby's nose and turn on the power.

**3** Say goodbye to nasal mucus and earwax.

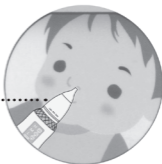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